F · S · T

EN

DE

CN

ES

FR

INSTRUCTION MANUAL HIGH TEMPERATURE BATTERY CAUTERY KIT

FINE SCIENCE TOOLS

HIGH TEMPERATURE BATTERY CAUTERY KIT, FST® ITEM NO. 18000-00 SAFETY INSTRUCTIONS

Before using this product read these instructions carefully and keep them in a safe place for future reference. The manufacturer takes no responsibility for misuse or any modification. Before use, always inspect all instruments for functionality and integrity. FAROMED GmbH Medizintechnik strictly prohibits any modification of the instrument. The CE-Sign will become invalid in case of use of any non-original spare part. This product may only be used by medical educated staff! Fused tips are covered by warranty if there is a material or manufacturing defect only.

RANGE OF APPLICATIONS

This cautery device can be used to stop bleedings and for coagulation of blood-vessels. Tips may be used with battery-handles and rechargeable-battery-handles only.


WARNINGS

- Do not use damaged instruments or equipment.
- Do not use this device in areas with flammable or explosive gases.
- Only authorised personnel may accomplish repairs.

CAUTION: BURN HAZARD!

The device is designed for short term applications (up to 5 min. per application) only, because it may become very hot during operation.

For intermittent operation only – no continuous use! Tips will become very hot during burning and the plastic tip-chip on the tip can start melting after 20 seconds of burning. Never lay down hot handle and tip onto or near the patient. Never place other devices or instruments over the handle. Prevent from any accidental operation, abuse and not intended use! Do not try to fix bended tips! Secondary use of not-cleaned tip may cause cross-infection! Clean immediately after use!

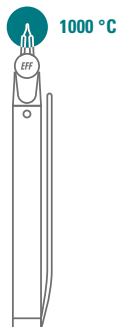
USAGE

New instruments and equipment have to be cleansed and disinfected before first use. Protect all parts of this product from any mechanical damage. Do not throw or apply force. For the cauterization of blood vessels on the scleroma, the cold tip is placed on the capillary bleeding spot. Remove the cautery simultaneously to switching on the current. Thus, before the cautery becomes heated, it has already been removed from the scleroma. This method, called "Contact with cold cautery", can be quickly acquired by the user after a short practice.

The total period of use depends on the charging status of the batteries. Due to the high current during use, the capacity of the batteries is running down fast. The tip has to be removed from the handle immediately after use to avoid damage to the tip and to prevent the cautery device to be switched on accidentally. Remove the batteries from the handle before storing the device for a longer time.

CHANGING OF BATTERIES AND PREPARING FOR OPERATION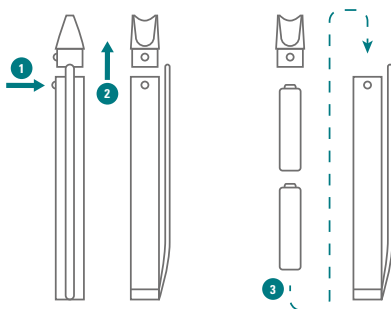


For changing the batteries, please proceed as shown. To make sure that the device is ready to use please check that the tip is fixated. Then press down the switch at the handle shortly. If the tip begins to glow, the device is ready for use. Do not use the tip without tissue contact because it may burn out, if the batteries are new.

CLEANSING/DISINFECTION AND STERILIZATION

Handle: The handle is supplied non-sterile. Depending on purpose of use it could be necessary to sterilize it before. Dismount the handle before cleansing and remove the batteries. Before cleansing and sterilisation please sort by stainless instruments, chromed instruments and affected instruments.

CLEANSING/DISINFECTION (HANDLE)

Wetting the handle completely by spray disinfectant with solution of "0,5% Kohrsolin FF". Exposure time: 15 min. Afterwards the handle has to rinse about 5 min. with demineralized water and dry with a lint-free cloth or compressed air. Don't use metallic brushes or any other abrasive cleaning tools. Dry instruments immediately after cleansing.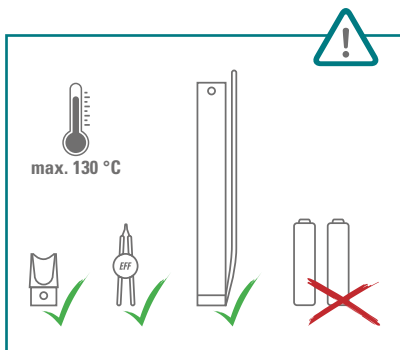

CLEANSING/DISINFECTION AND STERILIZATION

STERILIZATION (HANDLE)

ATTENTION: Before autoclaving ALWAYS REMOVE the batteries! After cleaning, steam sterilization at 134 °C with a hold time of 5 minutes is permitted only. Don't make use of hot sterilization! Always take care that no sediments are on the gold plated contacts at the switch of the handle. Residues at all contact areas can be reason for increase of electrical resistance which results in lower performance of tip and fast warming up of handle. Make sure that the switching lever is not bent so that the contact is almost permanently closed. It has to be easily bent back to a distance of the contacts of minimum 2 mm.

TIPS: The cautery tips have to be cleaned from any residue. Not completely removed residues can burn into the tips, which makes a cleansing impossible and the tip useless. The cautery tips can be disinfected by fluids as well as by autoclave. The platinum tip itself is sterile by the process of glowing a few seconds. The tips are supplied non-sterile. New tips have to be disinfected or sterilized before first use.

CLEANSING/DISINFECTION AND STERILIZATION

CLEANSING/DISINFECTION (TIPS)

Place tips into a strainer and dip into Sekusept Plus 4% for 15 min. Afterwards rinse with demineralized water for 5 min. Dry with cloth.

STERILIZATION (TIPS)


After cleaning, steam sterilization at 134 °C with a hold time of 5 minutes is permitted only.

STORAGE AND TRANSPORT CONDITIONS

- Transport and store in original packaging only.
- Protect the packing from humidity, contamination, extreme temperatures and UV-beam.
- Do not store on floor.
- Storage temperature: -30 to 60 °C
- Working temperature: 0 to 25 °C
- Humidity: 30 to 75%
- Barometric pressure: 700 to 1.060 hPa

DISPOSING

Used batteries, as well as the entire product and its equipment after its operating life, have to be disposed in compliance with the local regulations and law. The following applies for the European Union: According to the EU Directive 2002/96/EC on Waste Electronic and Electrical Equipment (WEEE) do not dispose of this product as unsorted municipal waste. Prepare this product for reuse or separate collection as specified by the directive. This directive does not apply, if this product is contaminated.

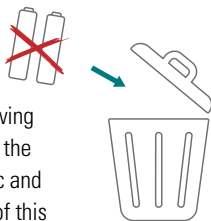TECHNICAL SPECIFICATIONS

Order Code	09-302 (with tips shape A and L)
	09-305 (with tips shape A, L, C, P, E)
Power supply	2x alkaline cell 1.5 V, type Mignon, LR6 (AA)
Temperature of tips	≈ 1000 °C
Weight (incl. batteries)	≈ 120 g
Application part	Type B
Operating environment	4 °C to 45 °C
Classification based on MPG	IIa

SPARE PARTS AND ACCESSORIES

Always use only original Duracell brand batteries (type Mignon, LR6). In case of using products of different brands the performance of device can differ. In addition, the liability of the manufacturer will expire.

00.1276 Tip of Platin-Iridium, Shape A

00.1278 Tip of Platin-Iridium, Shape C

00.1280 Tip of Platin-Iridium, Shape E

00.1277 Tip of Platin-Iridium, Shape L

00.1279 Tip of Platin-Iridium, Shape P

00.1272 Complete cautery handle
(without batteries)

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